




AEGIS LIFESCIENCES

PROTECTION THROUGH PERFECTION



SURGISPON[®] Anal Tampon

Absorbable Haemostatic Gelatin Sponge For Anorectal Use

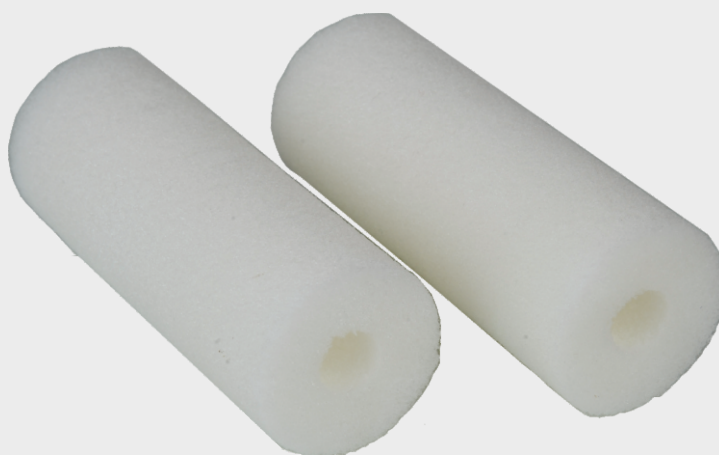

- Quick haemostasis in anorectal surgeries
- Best suitable for post operative haemostasis
- Painless and easy tampon discharge
- High blood absorption capacity
- Surgispon[®] Tampon is provided with an opening into which a flatus drain can be introduced

SURGISPON® Anal Tampon

Benefits/Advantages

Surgispon® Anal Tampon reduces post-operative pains more effectively than other known forms of anal tampons.

- High absorption capacity - up to 50 times of its own weight.
- Surgispon® anal gelatine sponge liquefies rapidly and is spontaneously expelled within 1 to 2 days.
- Used as a carrier for drugs like antibiotics, thrombin and haemostats glue.
- The gelatin sponge application can save up to 4 days of hospital stay for each rectal patient.
- Significantly increases haemostasis without increasing pain and discomfort.
- The immediate insertion of an absorbable haemostatic gelatin sponge can reduce the frequency of rectal bleeding.



Features

- Non-toxic
- Non allergenic
- Non-immunogenic
- Non-pyrogenic
- Biocompatible
- It has a non-water-soluble property
- Used dry or soaked with physiological solution
- Well adhering ability to the bleeding site
- Implantable and completely resorbable
- Does not stick to instruments

Surgispon® is manufactured from highly purified & ph neutral pharmaceutical gelatin foam of uniform porosity.

Surgispon® is single use & gamma sterilised absorbable haemostatic gelatin sponge.

Surgispon® anal Tampon is used in haemorrhoid operations and other surgical procedures in the anal, vaginal and rectal regions.

Painless and easy tampon discharge. The Surgispon® is easy expelled at first bowel movement, requiring no additional nursing intervention.

Where to use :

- Anorectal surgery – suitable to better facilitate post operative haemostasis.
- To achieve fast haemostasis in :
 - Haemorrhoidectomies
 - Anal fistula and fissures
 - Curettage and Hysterectomies
 - Surgery for rectal cancer
 - Rectal prolapse
 - Colorectal surgery
 - Insertion into rectum after transrectal prostate needle biopsy

How to use :

For rectal applications

In anal and rectal surgery, introduce a dry anal tampon with the help of a proctoscope at the end of the surgical procedure.

- Place dry gauze in the rima natis.
- The buttocks must be held together by means of a T bandage.
- Bacterial invasion ensures the rapid dissolution of the sponge which softens and is spontaneously removed by defecation.
- The Surgispon® Tampon is provided with an opening into which a flatus drain can be introduced.



After performing excision of hemorrhoidal piles, the operation is terminated by placing haemostatic absorbable gelatin sponge tampon in the anal canal.



For vaginal applications

Shape Surgispon® Tampon to fit speculum.
Insert, by means of a speculum a dry or soaked Surgispon® Tampon

Surgispon® Tampon may be used in combination with antibiotics, thrombin, chemo therapeutics without reduction of the haemostatic effect. Carefully read applicable instructions for use before usage.

'Biological Safety & Performance evaluation' has been performed on Surgispon® according to ISO 10993 and USP standards. A summary of these studies can be obtained on demand.



Product Code	Dimensions (mm)	No. of Pcs.	Packing
SSP-8030	80x30Ømm	06	Individual blister

AEGIS LIFESCIENCES PVT. LTD.

215/216, Mahagujarat Estate, Gam : Moraiya,
Tal. Sanand, Dist.: Ahmedabad-382213.

❖❖❖ **WC3P+RM** Changodar, Gujarat, India

M; +91 9512503131 | WhatsApp: +91 9512603131
E : marketing1@aegis-lifesciences.com
aegis.lifesciences@gmail.com